


EXTRACTING THE CONTENT OF THE QUIKREAD GO SOFTWARE FILE:


1. The downloaded software folder is archived and it needs to be extracted to a USB storage device. On your computer, navigate to your Downloads folder where you will find the downloaded software folder. Different browsers may have slightly different commands and names for archive folders.

Note: At any time of the process, do not change file or folder names. The name of the archive folder, as well as a later created file folder should have their name exactly as "qrzwupdate".


WRONG NAME. Must be exactly qrzwupdate

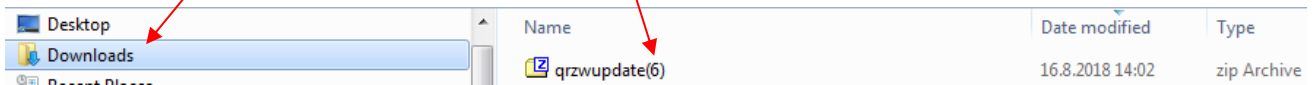


2. Go to the archive folder *qrzwupdate* and when your browser is on the line, right click. Depending on computer, you will get something like "Extract all" or like here, 7-zip and then *Extract files*. Extract the files. Some computers might not have zip files but have rar files.

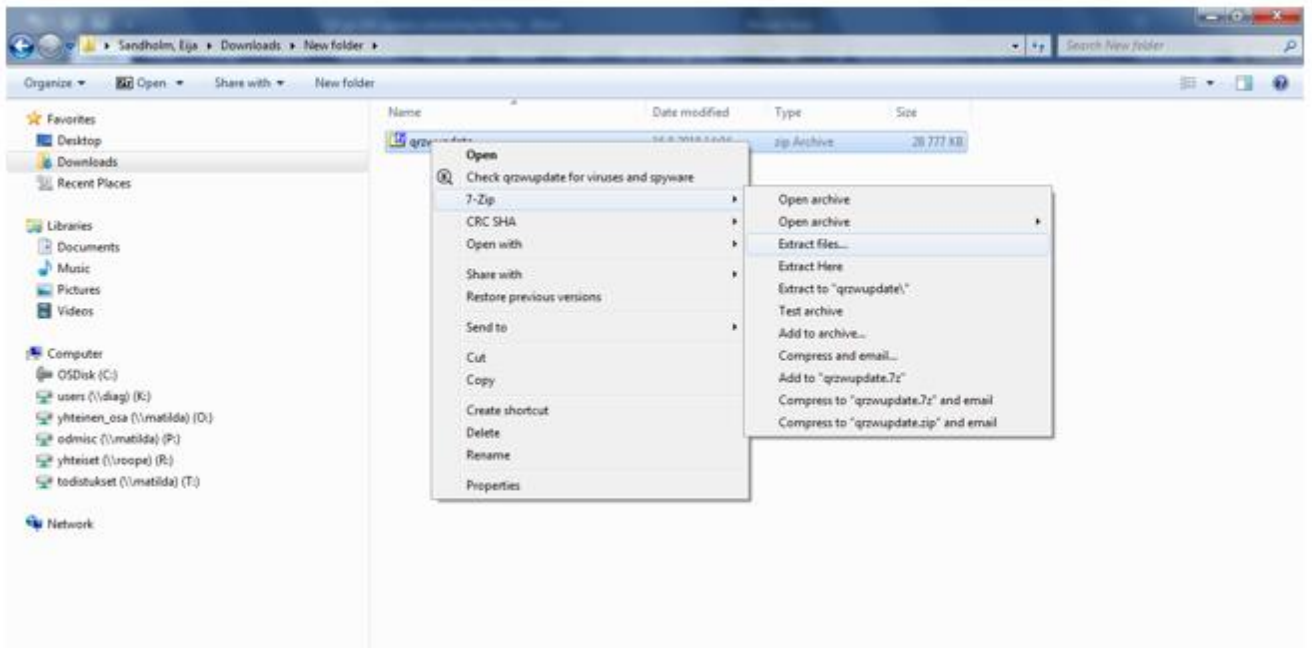


3. When you have selected *Extract files*, you will get this kind of screen. Select the three dots

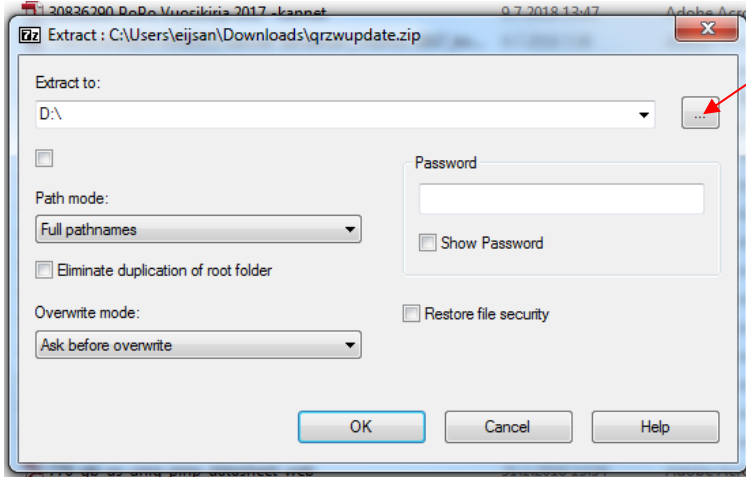


and find the USB storage device on your desktop.

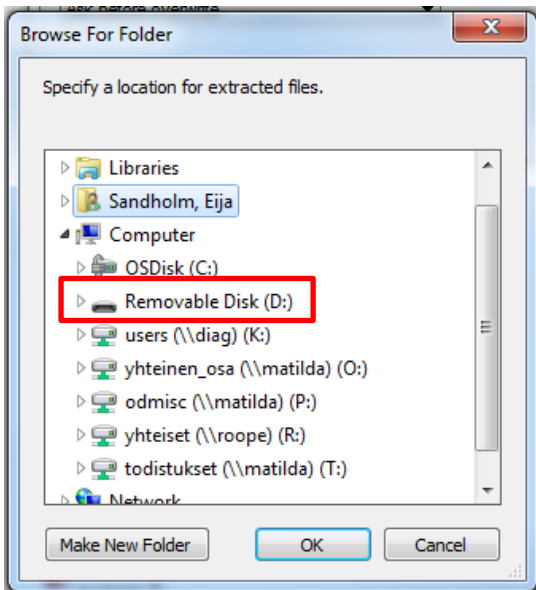

NOTE This field must be empty (see above) before starting to extract.

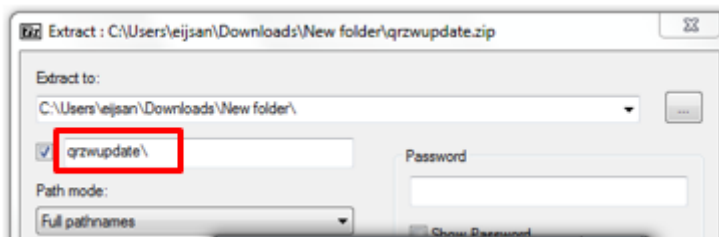

Then select OK and OK. A folder *qrzwupdate* will be created and the files extracted to your USB to that folder. When this is ready, you can use the USB for updating the software version on your QuikRead go instrument.

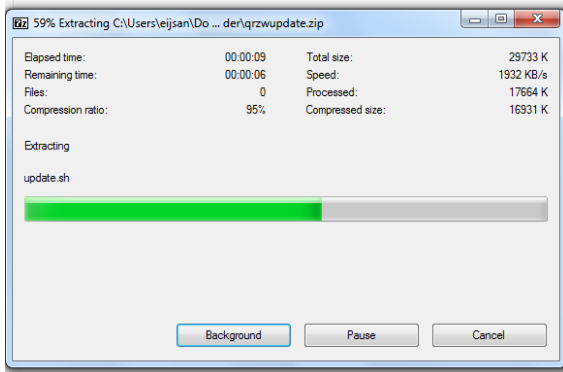

IMPORTANT! The extracted software files need to be in a File folder *qrzwupdate*, otherwise QuikRead go instrument cannot find them on the USB storage device.

Name	Date modified	Type
 qrzwupdate	31.1.2019 10:54	File folder