QuikRead go Plus software update

QuikRead go Plus software can be updated to a newer version via user interface. Download the newest software version to a USB stick at softwareupdate.quikread.com, or order a QuikRead go Plus software USB stick (prod. no. 155433).

NOTE: If rechargeable battery is in use, please remove it prior to updating software.

1. Connect the power supply cable and turn the QuikRead go Plus Instrument on. Wait for the Start-up to finish and the Home screen to appear.
2. Choose **Menu** → **Maintenance** → **Software update** to start updating.
3. The instrument asks you to insert a USB stick. Insert a USB stick with the software update file or the QuikRead go Plus software USB stick to one of the USB-ports and choose **Done**.
NOTE: When using downloaded update files please follow closely the instructions found on the software update website.

Instructions for use • English

4. The instrument checks if the USB stick contains a valid QuikRead go Plus software. This normally takes less than one minute. The version number of the software version on the stick and the one currently in use will be shown on the display. The instrument asks for confirmation before starting the update. If you agree, choose **Yes**.
 - In case there is an older software version on the USB stick a message *The USB has old software version* appears on the display. Choose **OK** to return to the Home screen.
5. QuikRead go Plus Instrument transfers the new software version from the USB stick and finalizes the update. QuikRead go Plus Instrument automatically restarts and returns to the Home screen.
NOTE: Do not remove the USB stick during the update.
Do not shut down QuikRead go Plus Instrument or unplug the power supply cable during the update.
6. After restarting it is safe to remove the USB stick.

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